# **plusmed**

health management

Model: **pM-2012 Plus**

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## Standard Features

- Simple to operate.
- Portable and lightweight.
- Pad consists of 135 bubble cells.

## Operating Instructions

1. Position pump on horizontal surface or hang it by means of the hook.
2. Open mattress on the bed. Secure it with end flaps or straps. Air tubes must be at foot end of bed.
3. Connect air feed tubes to air outlet on the mattress and pump, ensuring that the tubes are not “kinked” or twisted. Allow space for bed rails to drop freely.
4. Plug the pump into a suitable electrical power source and switch on. The switch indicator should now be illuminating.
5. Allow 20 minutes for mattress to inflate fully.
6. Cover with a sheet and tuck loosely.
7. Please use pressure adjustable knob to give maximum patient comfort.

## Technical Specifications

Pump	Mattress
Item: pM-2012 Plus Pump	Item: pM-2012 Plus Mattress
Power supply: AC220/230V 50Hz	Size: 200cm(L) x 90cm(W) x 7cm(H) (after inflation)
Air output: 4.5-5.5 liter/min.	
Pressure range: 110 mmHg - 130mmHg	135 bubble cells
Cycle time: 10 minutes	Without vents and with end flaps
Power consumption: 10 W	Material: PVC (EN-71, low toxic)
Fuse: 0.5A	Color: beige
Size: 22cm(L) x 12cm(W) x 8.5cm(H)	Repair kit included
Weight: 720 g	Maximum Patient Weight: 135 kg

## Cleaning Guidelines

- Pad can be cleaned by disinfectant solution.
- Do not immerse the pump in water.
- Do not heat or steam in autoclave.

*Warranty for Pump and Mattress: 2 year*

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